

H-Shaped Safety Belt - User Manual

1. Standard assembly – for general users



1.1 Place H-shaped safety belt on the back upholstery ◦



1.2 Pull up seat. Put the safety belt over cross-bar.



1.3 Buckle two ends of safety belt.



1.4 Released front buckle. Allow user to get seated. Adjust tightness of front buckle.



1.5 Adjust tightness of back buckles.

2. Reinforced assembly – for better support



1.1 Place H-shaped safety belt on the back upholstery ◦



1.2 Pull up seat. Put the safety belt over cross-bar.



1.3 Buckle two ends of safety belt in a cross.



1.4 Released front buckle. Allow user to get seated. Adjust tightness of front buckle.



1.5 Adjust tightness of back buckles.